PHYSIOSELECT® TCR

Zahnformenkarte

Tooth mould guide

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Carta de formas dentales

Tandvormenkaart

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HIGH DIVERSITY OF YOUNG,
AUTHENTIC MOLDS FOR A NEW
ATTITUDE TO LIFE.

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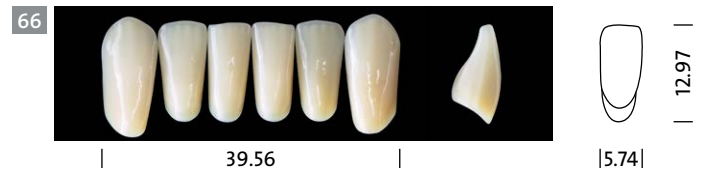
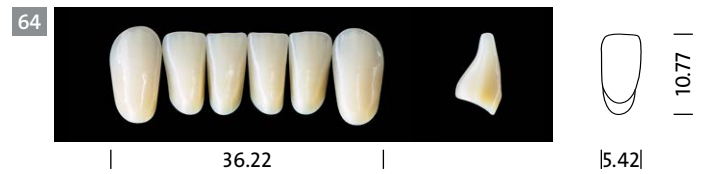
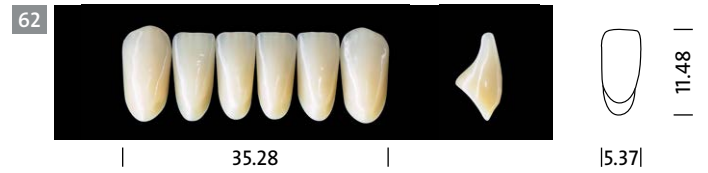
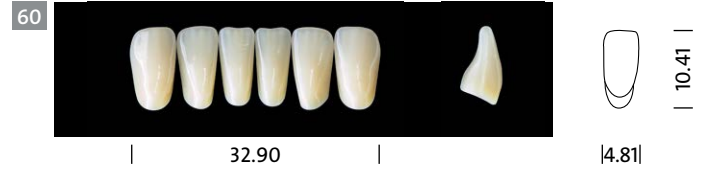
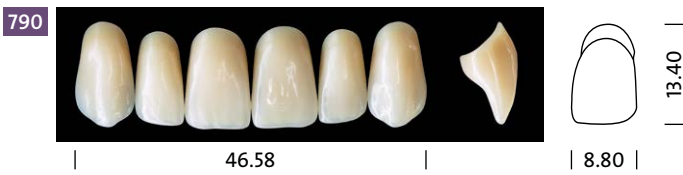
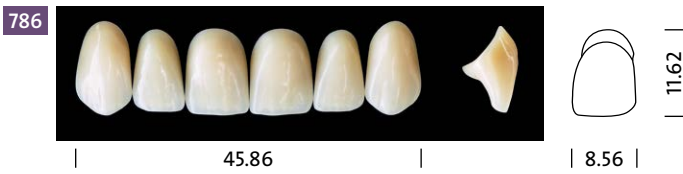
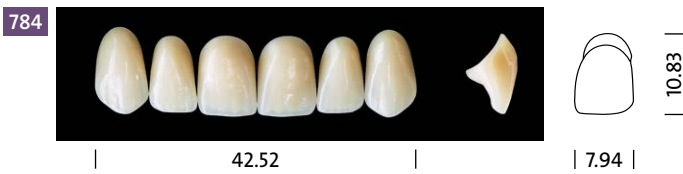
WWW.TOOTHSCOUT.COM

The App is an aid for dentists, dental technicians and prosthodontists, which facilitates the selection of tooth molds. This allows selection of suitable, esthetic teeth with a natural look for every patient.

And it is so simple.

Measure the width of the alar base using a Sliding Caliper, enter the data in the ToothScout App and choose from a variety of tooth molds. We give you the correct references of maxillary molds to mandibular molds and the matching posterior teeth. Whether tooth-to-two-tooth or tooth-to-tooth setup, everything is available.

The selected tooth molds can be saved under the created patient profile and the order enquiry can be sent directly from the app to the selected dealer by e-mail.

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Processing instructions for CANDULOR denture teeth

It must be ensured that the stability of the teeth and the correct shade effect are maintained. It is therefore recommended to observe the minimum layer thickness.

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To ensure that removable dentures remain functional over longer periods of time, the following prerequisites should be met:

- > **Dental materials may only be processed by dental professionals.**
- > **Fabrication in the dental laboratory according to the principles of best practice.**
- > To ensure a reliable material bond between PMMA/composite teeth and the denture base material, clean the teeth with steam, roughen them and apply monomer. When using an auto-curing polymer for finishing, it is recommended to create additional mechanical retentions.
- > The teeth should not come into contact with a flame or direct heat (for example alcohol torch).
- > Heat development as a result of too much pressure during grinding must be prevented.
- > Contact with solvents must be prevented.

CANDULOR AG
Boulevard Lilienthal 8
CH-8152 Glattpark (Opfikon)
T +41 (0)44 805 90 00
F +41 (0)44 805 90 90
candulor.com
candulor@candulor.ch

CE 0123

EC REP

CANDULOR Dental GmbH
Am Riedergraben 6
DE-78239 Rielasingen-Worblingen

CANDULOR Dental GmbH
Am Riedergraben 6
DE-78239 Rielasingen-Worblingen